GE Medical Systems

Technical Publications

**Vivid *i* SafeLock Cart User's
Manual - Multi-Language -
Volume 2**

CE 0344

**Supplement to Vivid *i*/Vivid *q* User
Manual**

Direction: 5410856

Revision: 01

Operating Documentation

Copyright © 2010 By General Electric Co.

GE Medical Systems

MANUAL STATUS

5410856

08-Dec-2010

DOC ID: DOC0886413

© GE Medical Systems. All rights reserved. No part of this manual may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means, electronic, mechanical, photocopying, recording, or otherwise, without the prior written permission of GE Medical Systems.

COMPANY DATA

GE Medical Systems Israel Ltd.

4 Etgar Street

39120 Tirat Carmel, Israel

Tel: (+972) 4851 9555 Fax: (+972) 4851 9500

GE Medical Systems Information Technologies GmbH,

Munzinger Strasse 5 D-79111 Freiburg, Germany

Tel: (+49) 761 45 43 0 Fax: (+49) 76145 43 233

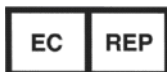
Table of Contents

Vivid <i>i</i> SafeLock Cart User Manual (ENG)	1
Vivid <i>i</i> turvalukuga käru SafeLock Cart kasutusjuhend (EST)	29
Vivid <i>i</i> SafeLock Cart Felhasználói kézikönyv (HUN)	59
Užívateľská príručka k vozíku Vivid <i>i</i> SafeLock Cart (SLO) .	89
Užívateľská příručka vozík Vivid <i>i</i> SafeLock (CZE)	121
Vivid <i>i</i> SafeLock Cart Kullanım Kılavuzu (TUR)	149
Ръководство на потребителя за количка SafeLock за Vivid <i>i</i> (BLG).....	179
Manualul Utilizatorului Căruciorului SafeLock pentru Vivid <i>i</i> (ROM)	211
Korisnički priručnik za Vivid <i>i</i> SafeLock kolica (CRO)	241
„Vivid <i>i</i> SafeLock“ vežimėlio naudotojo vadovas (LIT)	269
Vivid <i>i</i> SafeLock Cart lietotāja rokasgrāmata (LAT).....	299
Vivid <i>i</i> SafeLock uputstvo za korišćenje kolica (SER).....	329
Uporabniški priročnik za Vivid <i>i</i> SafeLock Cart (SVN)	359

Table of Contents

GE Medical Systems

Technical Publications

Vivid *i* SafeLock Cart User Manual

CE 0344

Supplement to Vivid *i*/Vivid *q* User Manual

Direction: 5410856-100

Revision: 01

Operating Documentation

Copyright © 2010 By General Electric Co.

GE Medical Systems

MANUAL STATUS

5410856-100

08-Dec-2010

© GE Medical Systems. All rights reserved. No part of this manual may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means, electronic, mechanical, photocopying, recording, or otherwise, without the prior written permission of GE Medical Systems.

COMPANY DATA

GE Medical Systems Israel Ltd.

4 Etgar Street

39120 Tirat Carmel, Israel

Tel: (+972) 4851 9555 Fax: (+972) 4851 9500

GE Medical Systems Information Technologies GmbH,

Munzinger Strasse 5 D-79111 Freiburg, Germany

Tel: (+49) 761 45 43 0 Fax: (+49) 76145 43 233

Introduction

- **Introduction** **3**
 - Attention4
 - Main Features5
 - Safety5
 - Additional safety information7
 - Finding information7
 - Contact Information8
- **Cart Components** **13**
 - Legend for all figures on the pull-out sheet13
- **Using the cart** **15**
 - Height adjustment15
 - Locking the wheels15
 - Mounting the Vivid i/Vivid q to the cart16
 - Releasing the Vivid i/Vivid q from the mounting platform19
- **Placing peripheral devices on the cart** **21**
 - Mounting a VCR on the cart22
- **Moving the cart with mounted system** **23**
 - Moving short distances23
 - Moving longer distances24
 - Transporting the cart by vehicle26
- **Maintenance and Service** **27**
 - Care and maintenance27
 - Electrical safety tests28
 - Booting from DVD - To Upgrade software28

Introduction

The Vivid *i* SafeLock Cart is a simple solution for mounting the Vivid *i*/Vivid *q*, for portable use in the clinic, providing storage for some of the most popular peripherals and accessories used day-to-day in Cardiology. The cart enables easy maneuvering of the system for improving accessibility and optimizing positioning.

Attention

This manual is a supplement to the Vivid *i*/Vivid *q* User Manual direction 5400907.

Read and understand all instructions in the Vivid *i*/Vivid *q* User Manual and in this Supplement SafeLock Cart User Manual, before attempting to use the Vivid *i* SafeLock Cart.

This manual has to be used in connection with the Vivid *i*/Vivid *q* User Manual.

All information contained in the Vivid *i*/Vivid *q* User Manual is relevant.

Keep the Vivid *i*/Vivid *q* User Manual and this supplement manual with the equipment at all time.

Periodically review the procedures for operation and safety precautions.

Main Features

- Small foot-print (50 x 50 cm - 19.8" x 19.8")
- Hand rest for extensive operation
- DVD/CD-R built-in to a special compartment
- Height adjustment (75 to 95 cm - 30" to 38") allows comfortable use while standing or sitting
- 5" swivel-lock wheels with front-brakes for easy transport
- 6 probe holders (3 on each side), 2 gel bottle holders (1 on each side), allow comfortable scanning on either side of the system
- 2 storage compartments for accessories storage (the height of one of them is adjustable)
- Local Area Network Connectivity (isolated)
- 3 isolated AC outlets for the peripherals/ accessories (total power of 150 VA)
- Cart mains support 100 - 120 VAC (250VA) or 220 - 240 VAC (250VA) (factory configured)
- 4 USB connectors for connecting peripherals/accessories
- Security cable for locking the Vivid *i*/Vivid *q* system to the SafeLock cart.
- EMC/IEC 601 certified

Safety

Please review Device Labels and Label Icon Descriptions in the Safety chapter of the Vivid *i*/Vivid *q* User Manual.

DANGER

Equipment is not suitable for use in the presence of FLAMMABLE ANESTHETIC MIXTURE with AIR or with OXYGEN or NITROUS OXIDE.

DANGER

To ensure proper grounding, connect this equipment to a receptacle marked "HOSPITAL ONLY" or "HOSPITAL GRADE".

Introduction

WARNING

Always lock the SafeLock Cart in its parked (locked) position after moving the system. Failure to do so could result in equipment damage.

WARNING

Equipment damage could result if special care is not taken when transporting the Vivid i/Vivid q system (and SafeLock Cart if applicable) in a vehicle.

CAUTION

**Electric shock hazard. DO NOT REMOVE COVER.
Refer servicing to qualified servicing personnel.**

CAUTION

When the Vivid i/Vivid q scanner is used with the SafeLock Cart, make sure the unit is firmly secured in the correct position on the cart otherwise it might become unstable and tip over.

CAUTION

Whenever the SafeLock Cart is to be moved along any incline, be careful and make sure that the Vivid i/Vivid q scanner and all peripherals are securely mounted on and secured to the cart before attempting to move it.

CAUTION

United States law restricts this device to sale or use by, or on the order of a physician.

SafeLock Cart Labels

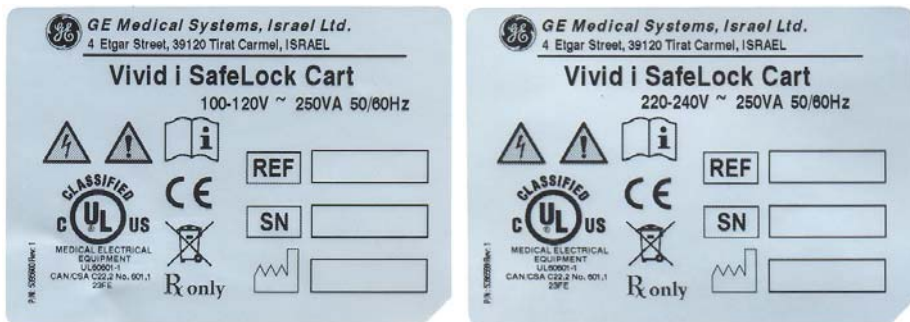
Figure 1-1: SafeLock Cart Labels

Additional safety information

Read all safety information located in Chapters 1 and 2 dealing with Safety in the Vivid *i*/Vivid *q* User Manual, Direction 5400907.

In this chapter the following topics are dealt with:

- Owner responsibility
- Important safety considerations
- Regulatory information
- Standards used
- Device labels and label icon description
- Classifications
- Acoustic Output
- ALARA
- System controls affecting acoustic power
- Patient safety
- Mechanical hazards
- Electrical hazard
- TEE probe safety
- Personnel and equipment safety
- Electrical safety
- Allergic reactions to latex-containing medical devices
- Electromagnetic compatibility (EMC)

And other safety information

Finding information

Table of Contents, lists the main topics and their location.

List of Figures, lists all figures and their location.

Headers and Footers, gives the chapter name and page number.

Contact Information

For additional information or assistance, please contact your local distributor or the appropriate support resource listed on the following pages:

INTERNET

<http://www.gehealthcare.com>
http://www.gehealthcare.com/usen/ultrasound/products/probe_care.html

USA

GE Healthcare TEL: (1) 800-437-1171
Ultrasound Service Engineering FAX: (1) 414-721-3865
9900 Innovation Drive
Wauwatosa, WI 53226

Clinical Questions

For information in the United States, Canada, Mexico and parts of the Caribbean, call the Customer Answer Center
TEL: (1) 800-682-5327 or (1) 262-524-5698
In other locations, contact your local Applications, Sales or Service Representative.

Service Questions

For service in the United States, call GE CARES
TEL: (1) 800-437-1171
In other locations, contact your local Service Representative.

Accessories Catalog Requests

To request the latest GE Accessories catalog or equipment brochures in the United States, call the Response Center
TEL: (1) 800-643-6439
In other locations, contact your local Applications, Sales or Service Representative.

Placing an Order

To place an order, order supplies or ask an accessory-related question in the United States, call the GE Access Center
TEL: (1) 800-472-3666
In other locations, contact your local Applications, Sales or Service Representative.

CANADA	GE Healthcare TEL: (1) 800-664-0732 Ultrasound Service Engineering 9900 Innovation Drive Wauwatosa, WI 53226 Customer Answer Center TEL: (1) 262-524-5698
LATIN & SOUTH AMERICA	GE Healthcare TEL: (1) 262-524-5300 Ultrasound Service Engineering 9900 Innovation Drive Wauwatosa, WI 53226 Customer Answer Center TEL: (1) 262-524-5698
EUROPE	GE Ultraschall TEL: 0130 81 6370 toll free Deutschland GmbH & Co. KG TEL: (33) 130.831.300 Beethovenstrasse 239 FAX: (49) 212.28.02.431 Postfach 11 05 60 D-42655 Solingen
ASIA	GE Ultrasound Asia (Singapore) TEL: 65-291 8528 Service Department - Ultrasound FAX: 65-272-3997 298 Tiong Bahru Road #15-01/06 Central Plaza Singapore 169730
JAPAN	GE Healthcare Japan Corporation TEL: (81) 42-648-2910 Customer Service Center FAX: (81) 42-648-2905
ARGENTINA	GEME S.A. TEL: (1) 639-1619 Miranda 5237 FAX: (1) 567-2678 Buenos Aires - 1407
AUSTRIA	GE GesmbH Medical Systems Austria TEL: 0660 8459 toll free Prinz Eugen Strasse 8/8 FAX: +43 1 505 38 74 A-1040 WIEN TLX: 136314
BELGIUM	GE Medical Systems Benelux TEL: 0 800 11733 toll free Gulkenrodestraat 3 FAX: +32 0 3 320 12 59 B-2160 WOMMELGEM TLX: 72722

Introduction

BRAZIL	GE Sistemas Medicos TEL: 0800-122345 Av Nove de Julho 5229 FAX: (011) 3067-8298 01407-907 Sao Paulo SP
CHINA	GE Healthcare - Asia TEL: (8610) 5806 9403 No. 1, Yongchang North Road FAX: (8610) 6787 1162 Beijing Economic & Technology Development Area Beijing 100176, China
DENMARK	GE Healthcare TEL: +45 43295400 Park Alle 295 FAX: +45 4329 5399 DK-2605 Brøndby
FRANCE	GE Medical Systems TEL: 05 49 33 71 toll free 738 rue Yves Carmen FAX: +33 1 46 10 01 20 F-92658 BOULOGNE CEDEX
GERMANY	GE Ultraschall TEL: 0130 81 6370 toll free Deutschland GmbH & Co. KG TEL: (49) 212.28.02.207 Beethovenstrasse 239 FAX: (49) 212.28.02.431 Postfach 11 05 60 D-42655 Solingen
GREECE	GE Medical Systems Hellas TEL: +30 1 93 24 582 41, Nikolaou Plastira Street FAX: +30 1 93 58 414 G-171 21 NEA SMYRNI
ITALY	GE Medical Systems Italia TEL: 1678 744 73 toll free Via Monte Albenza 9 FAX: +39 39 73 37 86 I-20052 MONZA TLX: 3333 28
LUXEMBOURG	TEL: 0800 2603 toll free
MEXICO	GE Sistemas Medicos de Mexico S.A. de C.V. Rio Lerma #302, 1° y 2° Pisos TEL: (5) 228-9600 Colonia Cuauhtemoc FAX: (5) 211-4631 06500-Mexico, D.F.

NETHERLANDS	GE Medical Systems Nederland B.V. TEL: 06 022 3797 toll free Atoomweg 512 FAX: +31 304 11702 NL-3542 AB UTRECHT
POLAND	GE Medical Systems Polska TEL: +48 2 625 59 62 Krzywickiego 34 FAX: +48 2 615 59 66 P-02-078 WARSZAWA
PORTUGAL	GE Medical Systems Portuguesa S.A. TEL: 05 05 33 7313 toll free Rua Sa da Bandeira, 585 FAX: +351 2 2084494 Apartado 4094 TLX: 22804 P-4002 PORTO CODEX
RUSSIA	GE VNIIEM TEL: +7 495 739 6931 18C, Krasnopresnenskaya nab. FAX: +7 495 739 6932 123317 MOSCOW
SPAIN	GE Healthcare TEL: +34 91 663 25 00 Edificio Gobelas I FAX: + 34 91 663 25 01 C/ Gobelas 35-37 E-28023 Madrid
SWEDEN	GE Medical Systems TEL: +46 (0)8 - 559 500 00 FE 314 FAX: +46 (0)8 - 559 501 24 S-171 75 Stockholm
SWITZERLAND	GE Medical Systems (Schweiz) AG TEL: 155 5306 toll free Sternmattweg 1 FAX: +41 41 421859 CH-6010 KRIENS
TURKEY	GE Healthcare, Turkiye TEL: +90 212 366 29 00 Sun Plaza FAX: +90 212 366 29 99 Dereboyu Sok. No 24/7 34398 Maslak ISTANBUL

Introduction

**UNITED
KINGDOM**

GE Medical Systems TEL: 0800 89 7905 toll free
Coolidge House FAX: +44 753 696067
352 Buckingham Avenue
SLOUGH
Berkshire SL1 4ER

**OTHER
COUNTRIES**

NO TOLL FREE TEL: international code + 33 1 39 20 0007

Manufacturer

GE Medical Systems, Israel, Ltd.
4 Etgar st.
39120 Tirat Carmel
Israel
Tel.: (+972) 4851 9555, Fax: (+972) 4851 9500

Chapter 1

Cart Components

Legend for all figures on the pull-out sheet

Number	Description
1	Hand rest
2	System release handle
3	Upper storage shelf with printer
4	Swivel-lock front wheel with foot brake Two-position brake, as follows:
5	Lower lever - Brake and swivel-lock lever Upper lever - Brake and swivel release lever
6	Cable holder used for wrapping power cable
7	Mounting platform
8	Probe, cable & gel holders: situated on either side of the upper storage shelf
9	Adjustable telescopic cart column
10	Strapping rails, for mounting the various peripherals
11	Lower storage shelf for VCR or other heavy peripherals
12	Foot rest
13	Network insulator connection
14	Main shelf height-lock handle, for raising or lowering the main shelf

Cart Components

Number	Description
15	Four non-isolated USB connectors for various peripherals
16	DVD-CD-R, built-in
17	Three AC power outlets for peripherals
18	Cable management opening on rear of shelf, for cables to peripheral on lower shelf
19	Brake release lever for front-wheels break & swivel
20	Anti-theft cable
21	Rear locking-latch spring-loaded - released by Release handle
22	Sliding guides - rear
23	Front locking-tooth
24	Sliding guides - front
25	USB cable to system
26	Power cable to system
27	Power cable storage hook
28	Gel-bottle holder
29	Probe cable holder
30	Strap for mount peripherals
31	Potential equalization grounding connection
32	Power cable connection
33	Circuit breaker
34	Rear wheel swivel
35	Power ON green LED indicator
36	Vivid <i>i</i> /Vivid <i>q</i> Carrying handle

Chapter 2

Using the cart

Height adjustment

To adjust the height of the mounting-platform raise the Main shelf height-lock handle (item 14) and pull the platform up or push it down with both hands.

When the lever is released the platform remains at the adjusted height.

Locking the wheels

To lock the front-wheels, with your foot press down the Brake lock lever (see Figure 2-1, item 5). Both the wheel brake and swivel are now locked.

To unlock the wheels, with your foot press down the Brake release lever (see Figure 2-1, item 19).

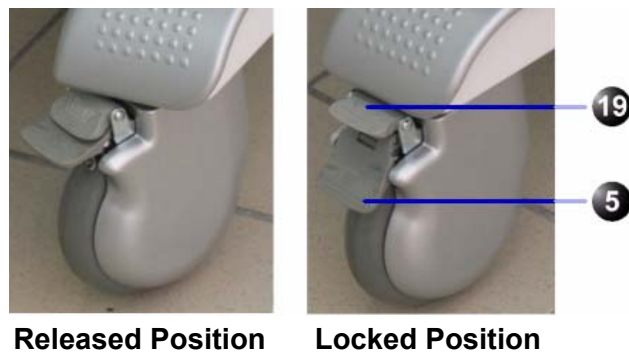
Figure 2-1: Wheel locking

Using the cart

Mounting the Vivid *i*/Vivid *q* to the cart

The Vivid *i*/Vivid *q* can be mounted either while the display screen is closed (system is fully shut-down or in standby mode) or while the display screen is open and the system is powered ON, OFF or in Standby.

NOTE: A battery or an empty battery-shell should always be attached to the system before mounting system to the cart.

CAUTION

Use caution when mounting system while it is turned-on, to avoid shocks or vibrations which may be harmful to the hard-drive.

To mount the Vivid *i*/Vivid *q* on the cart:

1. Tilt the Vivid *i*/Vivid *q* carrying handle upwards so that it does not interfere with the mounting procedure.
2. Hold the Vivid *i*/Vivid *q* slightly tilted towards you and place the front bottom part over the front sliding guides.

Figure 2-2: Placing on front sliding guides

Using the cart

3. Lower the rear part of the Vivid *i*/Vivid *q* to sit over the rear sliding guides. The system now rests over the four sliding guides.

Figure 2-3: Placing on rear sliding guides

4. While securing the mounting platform with your fingers to prevent the cart from moving, gently slide the Vivid *i*/Vivid *q* backwards until you hear a click.

Figure 2-4: Pushing to the rear

Using the cart

5. Gently pull the system upwards to verify that it is locked firmly in position.
Note: Before connecting the cables at the rear of the system, switch off the main circuit breaker at the rear of the SafeLock cart.
6. Connect the cables at the rear of the Vivid *i*/Vivid *q*. The cables to connect are listed below. The order of connection is not important:
 - L-Shaped circular plug of the power cable
 - Internal LAN network cable. (May be skipped if not using network connectivity.)
 - USB Cable - Connect the USB plug to the lower USB connector on the rear-panel of the Vivid *i*. This leaves convenient space for connecting other USB devices to the upper connector. (May be skipped if not using any USB peripherals.)
7. Switch on the main circuit breaker at the rear of the SafeLock cart.

NOTE: Check the Power-ON LED indicator beneath the USB connectors (item 15 and 35 in Figures) to verify connection to AC mains.

Once the system is mounted on the SafeLock cart, it can be locked to the SafeLock Cart using the security docking cable.

Insert the cable lock into the system's security slot and turn the key 90 degrees in any direction. To unlock, simply turn the key 90 degrees in the opposite direction.

Releasing the Vivid *i*/Vivid *q* from the mounting platform

The Vivid *i*/Vivid *q* can be dismantled from the cart either while display screen is closed (system is shut-down or in standby mode) or while the display screen is open and the system is either powered ON, OFF, or in Standby mode.

NOTE: Before dismantling the Vivid *i*/Vivid *q* while it is powered ON, be sure to check availability and charge of battery, as system will switch-over to battery operation as soon as it is released.

To release the Vivid *i*/Vivid *q* from the mounting platform:

1. Remove the cables from the rear of the Vivid *i*/Vivid *q*. This includes: Power plug, USB connector, and internal network LAN cable.
2. Place the palm of your hand on the cart's hand-rest and pull the release handle. With the other hand, slide the system towards you until the hand-rest stops it.

Figure 2-5: Pulling forward to release

3. Lift up the Vivid *i*/Vivid *q* system from the platform.

Chapter 3

Placing peripheral devices on the cart

NOTE: See the Vivid *i*/Vivid *q* User Manual for the list of approved peripherals, instructions on electrical connection, setting up, and using these peripherals.

Peripheral devices can be placed on both upper and lower storage shelves.

The DVD-CD writer can be placed in a dedicated compartment, as shown in item 16.

Place the heavy devices on bottom shelf, for improved stability.

Use any one of the three AC power outlets for peripherals (item 17) and USB ports (item 15) to connect devices to the system.

WARNING

In case a peripheral device is to be connected to an external power source, this device must be of medical grade.

Straps should be used to hold all devices in place on the cart.

When lowering the height of the cart, be sure to check that the upper storage shelf does not hit any device on the bottom shelf.

TIP: Use enough “Velcro” and “tie-wraps” to arrange the peripheral-cables in a convenient location at the rear of the shelves. Be sure to use the cable openings in the rear (item 18).

Placing peripheral devices on the cart

Mounting a VCR on the cart

As the VCR is rather heavy, it is important to place it on the lower rather than on the upper shelf.

To mount a VCR on the cart:

1. Place the VCR on the lower shelf (item 11).
2. Use “Velcro” to mount the Video scan-converter by stacking it over the VCR.

Chapter 4

Moving the cart with mounted system

Moving short distances

When planning to move the system only a short distance, such as to an adjacent room, the system can remain powered ON. If the expected move is over an area that will not allow a smooth ride, it is recommended to close the display and bring the system to “Standby” in order to protect it from excessive vibrations.

To move the cart with mounted system:

1. Release the brakes on both front wheels.
2. Disconnect the AC power-plug and wrap the cable around hook on rear handle.
3. Disconnect any other cables from the cart that are connected to permanent sockets in the room.
4. Make sure all probes are in their probe-holders.
5. Wrap-up ECG cable and all probe cables on cable-hooks so they do not touch floor or wheels.
6. Make sure all peripherals are held secure with straps.
7. Check that any non-secured items are not left in a position where they might fall.

Moving the cart with mounted system

Moving longer distances

When planning to move the system through long corridors and elevators, ensure of the following:

To move longer distances:

1. Close the LCD screen. This switches the system to "Standby" and protects the hard-disk from excessive vibrations that might cause damage while disk is in operation. This will also protect the LCD shaft from excessive vibrations.
2. Complete the previous procedure: "Moving short distances" on page 23.
3. Push the system by using the special handle on the rear of the cart. This will ensure balance and proper maneuvering.

Note: While cart is being moved pay special attention to the following:

1. The protruding ECG connector. Avoid bumping the connector into doors or beds.
2. The protruding power cord connector on the bottom rear of the cart. Avoid pushing the rear of the cart into a wall.

Note: The cart's latest version contains a protective support for the ECG connector and a wall-protection bumper to protect the AC cable, as shown in the following Figure 4-1 and Figure 4-2. These protective devices can be installed on carts that did not have them on the original installation.

Moving the cart with mounted system

Figure 4-1: Protective support for the ECG connector

Figure 4-2: Wall-protection for the AC cable

Moving the cart with mounted system

Transporting the cart by vehicle

When planning to transport the system by vehicle, follow all above steps. In addition:

1. Switch the system off.
2. Disconnect probes and place them in their appropriate packages.
3. Remove the Vivid *i*/Vivid *q* system from the cart and place in RollPac, carrying case, or other protective packaging.
4. Remove peripherals from upper and lower shelves. The DVD drive can remain in its compartment. Place peripherals in suitable packaging.
5. Lower the cart to its minimal height.
6. Place cart in upright in the vehicle and lock the wheels.
7. Use securing straps to secure the cart so that it does not move while traveling.
8. Do not place anything on the top tray of the cart, as this may damage some of the locking-hooks, pins or connectors located on this shelf.

CAUTION

When transporting the SafeLock cart in a vehicle, always secure it in an upright position.

Chapter 5

Maintenance and Service

Care and maintenance

- Check all four SafeLock Cart wheels and castors for wear.
- Check the brakes on each of the two front wheels. Make sure that pressing the brake locks the wheel, preventing it from swiveling or rolling and therefore stopping movement of the SafeLock Cart.
- Verify that releasing the brakes allows free movement of the SafeLock Cart.
- Check the SafeLock Cart raise/lower motion and verify proper operation of the locking and releasing mechanism.
- Inspect the cart for dents, scratches, or cracks.
- Inspect the probe holders for cracks or damage.
- Clean the probe holders with warm water and a damp cloth to remove all traces of gel (Soaking may be required to remove excess gel.)
- Check the power cord for cuts, loose hardware, tire marks, exposed insulation or other deterioration, and verify continuity. Tighten the clamps that secure the power cord to the unit and the outlet plug to the cord. Replace the power cord and clamp, as required.

Electrical safety tests

To perform electrical safety tests, follow instructions provided in the Vivid *i*/Vivid *q* Service Manual Direction R2423163-100, Section 10-7 - Electrical Safety Tests.

For further instructions for care and maintenance of the Vivid *i* see the Vivid *i*/Vivid *q* User Manual - Chapter 14.

Booting from DVD - To Upgrade software

When booting up from the DVD to load software updates, follow the connection procedure described below:

1. Disconnect the DVD cable from the cart's USB port (item 15).
2. Disconnect the USB cable at the rear of the Vivid *i*/Vivid *q* (item 25).
3. Connect the DVD USB cable directly to the lower USB port at the rear of the Vivid *i*/Vivid *q* system.
4. Perform boot-up/update as required.
5. After the software upgrading process, restore both USB plugs.

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Copyright © 2010 General Electric Co.
Copyright © 2010, General Electric Co.
P.O. Box 414, Milwaukee, Wisconsin 53201 U.S.A.
Deutschland GmbH & Co. KG FAX: 49 212.28.02.431