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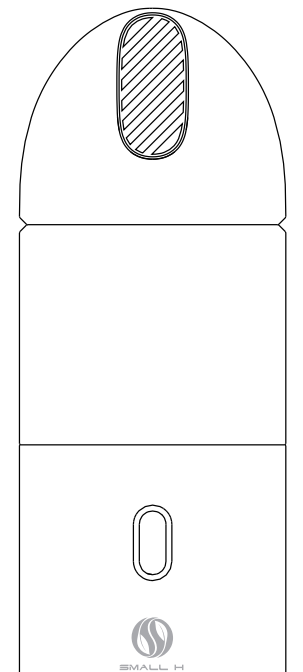
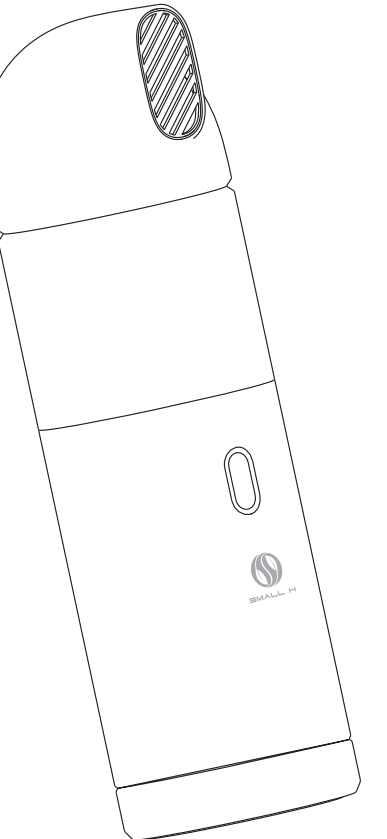
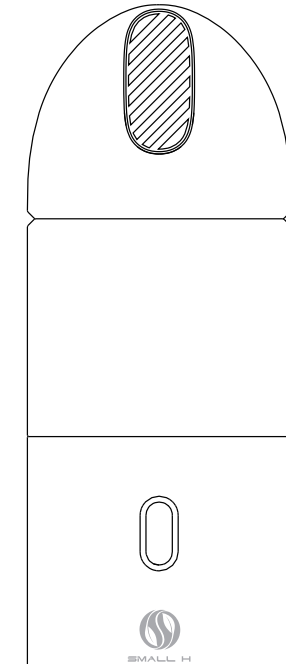
Small H Mini Hydrogen-Oxygen Generator D-HO-003 Operating Instructions


Small H Mini Hydrogen-Oxygen Generator is a portable device to generate hydrogen-oxygen mixed gas (H₂/O₂: 66.6% / 33.3%) via electrolysis of water. The device is developed with full independent intellectual property rights by Dalian Institute of Chemical Physics, Chinese Academy of Sciences and utilized with non-noble metal catalytic electrodes and hydrogen-oxygen aid. It is featured with small-sized (75 mm×75mm×200mm), lightweight (< 420 g), noiseless (< 30 dB), low-energy cost (≤ 15 W), multi-optional in breathing modes, easy-handling and built-in rechargeable battery.

SPECIFICATIONS


Model	D-HO-003
Rated voltage	DC 5 V
Rated current	3 A
Rated power	15 W
Gas output	20-30 mL/min (external adapter power supply) 10-30 mL/min (internal battery power supply)
Dimension	75 mm × 75 mm × 200 mm
Net weight	< 420 g

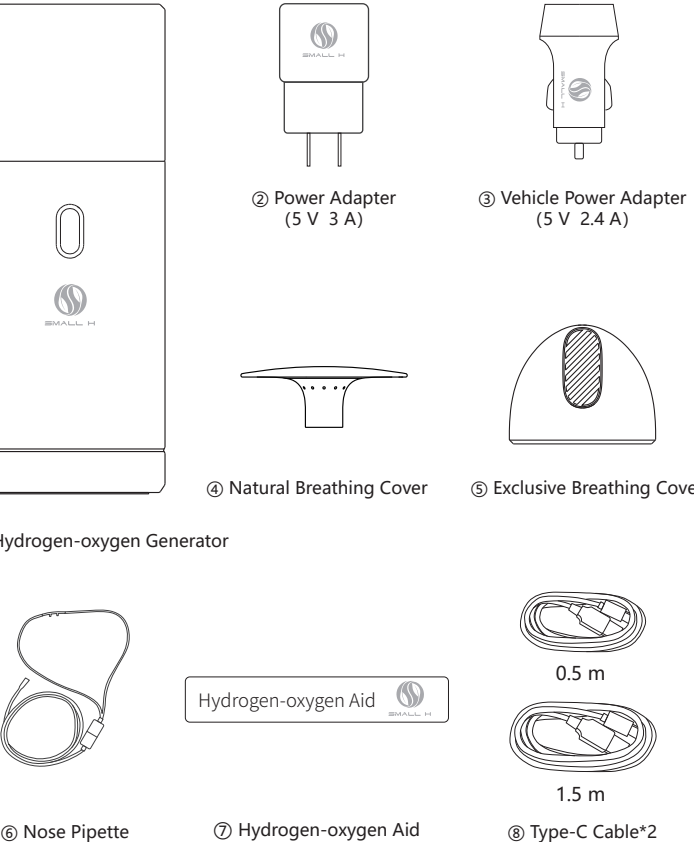PARTS LIST


DESCRIPTION

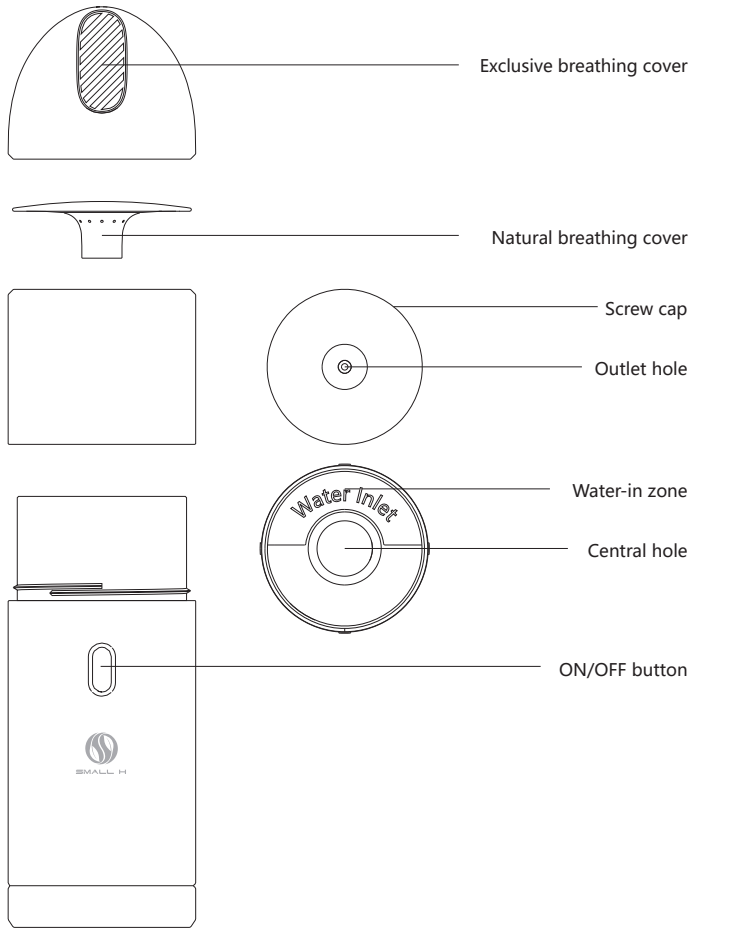
OPERATION MANUAL

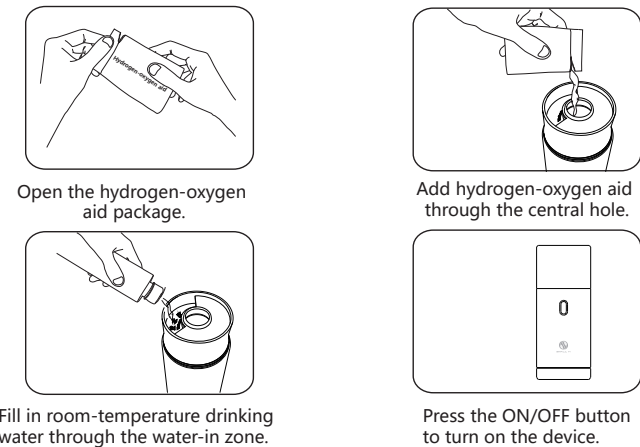
- Before use for the first time or after cleaning, add one packet (25 gram) of hydrogen-oxygen aid through the central hole, fill in about 200 mL room-temperature drinking water through the water-in zone (please stop filling when the water level reaches the maximum-volume mark) and screw the lid tightly.
- Connect the power supply. Press and hold the ON/OFF button for 1.5 seconds to turn on the device, then indicator is lighted with a beep sound and the device starts working.
- With the insufficient water level during device working, the indicator will keep flashing accompanied with "beep-beep-beep" sound, reminding for adding water. Please unscrew the lid and add about 150 mL room-temperature drinking water through the water-in zone (please stop filling when the water level reaches the maximum-volume mark), and the device will continue the previous working mode. Otherwise, without water-adding in time, the device will automatically shut down after 5 minutes.
- When using the built-in battery, the indicator light is green if the battery is fully charged, but it will turn red to remind charging when the battery power is low.
- Please press and hold the ON/OFF button for 1.5 seconds to turn off the device, and indicator light will go off.
- The battery can be charged after the device shutdown, and the indicator lights up green and flashes slowly during charging, and the indicator lights up green when the battery is fully charged.


USER SCENARIO

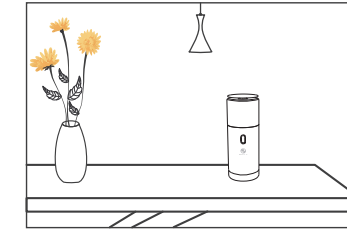① Natural breathing mode

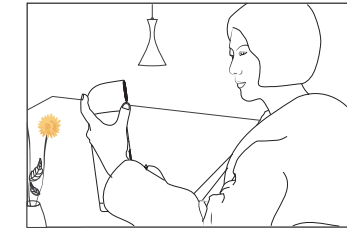② Exclusive breathing mode

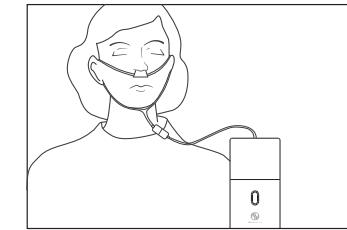③ Deep breathing mode

NOTES

In order to avoid personal injury or property damage caused by electric shock, over-temperature, fire, explosion, mechanical hazard, and inappropriate usage, please carefully read and apply the following instructions before using the device.


- Do not store this device and accessories in a humid and dusty environment. During use, keep the room ventilated. The device must be kept out of fire, liquid or rains; the device should be avoided hardware impact, casting, vibration or gas-leak covering so as not to damage the device.
- Do not tilt or invert the device to avoid spillage of the hydrogen-oxygen aid.
- The device is equipped with tipping protection, and it will be automatically turned off after 20 seconds if accidental tipping occurs during use.
- Do not block gas outlet.
- Though the hydrogen-oxygen aid is avirulent and insipid, it must be kept out of the reach of children to avoid eating by mistake.
- The device will be slightly heated during use, which is a normal phenomenon.
- The device is equipped with overheat protection. Please do not add hot water.
- Ensure the tightly connection between plug and socket.
- The attached adapter is the only warrant power supply to avoid damage to the device.
- Children under the age of 10 or persons with intellectual disabilities must use this device under the supervision of a guardian.
- The nose pipette is for exclusive use only to avoid cross-infection.
- Do not use the device in places with fire, heat or spark hazards.
- Attention to the spark from static electricity. Actions to generate static electricity is highly forbidden, such as taking off the sweater, comb hair, etc.
- The device is forbidden to use with negative-ion generators.
- Users must follow the safety regulations or laws. Do not use it in situations where it is prohibited by laws and regulations to avoid accidents.
- Any abnormal scenarios (such as smoke, smell, leakage, etc.) during use, please turn off the device and disconnect the power supply immediately and send the device to the official supplier for maintenance.
- Do not disassemble, modify or repair the device without official authority to avoid body injury. If there is any problem or service requirement, please contact the official supplier.
- The applicable altitude and climate conditions of this device are determined by the adapter. Please read the icons on the adapter before use.
- Do not directly dispose the device or use as children's toy so as not to cause safety hazards or environmental issue.

COMMON PROBLEMS AND SOLUTIONS

Problems and malfunctions	Reasons and solutions
Turn-on failure	Check the connection of power supply and cable; Check the status of internal battery.
Washing water turns black	For the first time of use, the catalytic electrodes in the device will be activated and generate black species, which is a normal phenomenon.
"Dth" sound from gas outlet	The sound is caused by friction between water vapor and gas outlet. Please use the swab to clean the gas outlet.
Battery Charging failure	It is not allowed to charge the device during it is working. Please make sure to turn off the device for battery charge.
Others	Please subscribe "XZ信孚实业" Wechat Public Channel.

MAINTENANCE

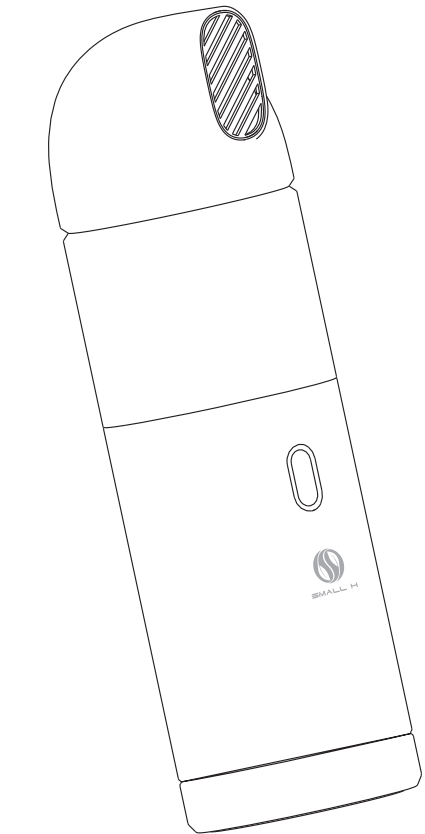- It is recommended to thoroughly wash and clean the interior and lids of using room-temperature drinking water every three months. Then add one package (25 grams) of fresh hydrogen-oxygen aid before use.
- For lids washing, use room-temperature drinking water to wash the lids and dried with swab.
- Warrant: 1 year
- Contact details:
Company: XZ Technology Industry (Shenzhen) Co., Ltd
Address: Floor 2, Building 14, Conceptual Incubation Space, Minzhi Street, Longhua District, Shenzhen, Guangdong, China.
Phone: 400-0755-817

Small H₂ Big Life